

# DtM Face Shield

## Info Sheet




REDUCE THE RISK OF EXPOSURE & PROTECT YOURSELF WITH DtM FACE SHIELD

Tool Gauge is a build-to-print manufacturer of complex metal parts and assemblies. Using our existing expertise and capabilities, we have manufactured a DtM Face Shield in response to the unprecedented shortages in Personal Protective Equipment (PPE).

The DtM Face Shield is comprised of an injection molded polypropylene headband, polycarbonate shield, and an elastic headband strap.

Tool Gauge is offering the DtM Face Shield at our cost using materials on hand or in our supply chain in order to move as rapidly as possible in scaling up production.

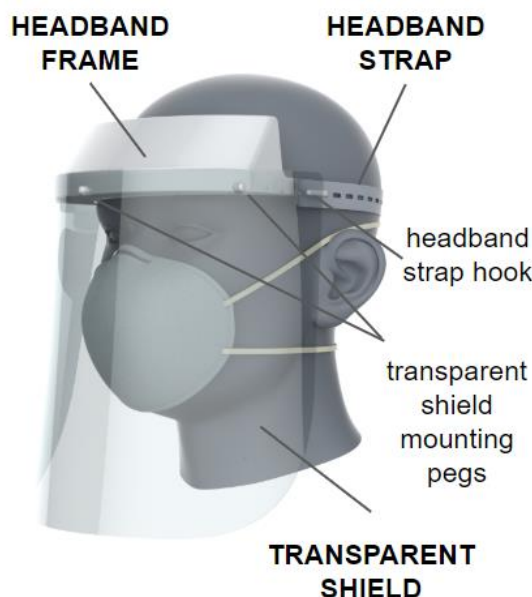

## Face Shield features

**UNIVERSAL FIT:** DtM Face Shield fits most head sizes with an adjustable headband.

**FULL LENGTH PROTECTION:** The DtM Face Shield provides 12x8 inch barrier to reduce the limit of infectious aerosol and liquid splatter.

**FLEXIBLE CONSTRUCTION:** Made from injection molded polypropylene with mounting pegs, polycarbonate shield, and elastic headband strap with hook.

**SCRATCH RESISTANT:** Made from injection molded polypropylene with mounting pegs, polycarbonate shield, and elastic headband strap with hook.

**CHEMICALLY RESISTANT:** DtM Face Shield is designed to be reused. Clean the face shield with Super Sani-Cloth, 10% chlorine bleach solution, CaviWipes, or soap and water.

**Precision Cut:** DtM Face Shield is designed for consistent fit & finish.

Submit Orders or Product Questions to:

[SALES@TOOLGAUGE.NET](mailto:SALES@TOOLGAUGE.NET)

This product has not been FDA cleared or approved. This device is being offered for the duration of the FDA Emergency Use Authorization for Face Shields (9 April, 2020). This product is only authorized for the duration of the declaration that circumstances justifying the authorization of emergency use under Section 564(b)(1) of the Act, 21 USC 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner. The use of this DtM face shield should always come secondary to existing PPE equipment, standards, and protocol options if available. The decision to implement this device should be made with consultation of the corresponding institution's occupational health and infection control departments and in line with CDC Strategies for Optimizing the Supply of Eye Protection.

# DtM Face Shield

## Info Sheet



### FREQUENTLY ASKED QUESTIONS

**HOW CAN I LEARN MORE ABOUT THE DtM Face Shield?** To learn about the DtM Face Shields, please contact [sales@toolgauge.net](mailto:sales@toolgauge.net) or visit.

**HOW CAN I ORDER DtM FACE SHIELDS?** Please send all order requests to [sales@toolgauge.net](mailto:sales@toolgauge.net). Please include your name, telephone number, physical address, email address, and business name (if applicable).

**HOW MUCH DOES A DtM FACE SHIELD COST?** The DtM Face Shield is available at cost of **\$3.46 USD**, plus shipping fees. Orders must be placed with a 200 minim order quantity or more to qualify.

**WHAT IS THE MATERIAL COMPOSITION OF THE DtM Face Shield?** The DtM Face Shield is made up of several components. The headband is injection molded polypropylene, the shield is polycarbonate, and an elastic head band with a strap hook.

**DO THE Dtm FACE SHIELDS COME ASSEMBLED?** No, the DtM Face Shields come unassembled. Each face shield is packaged individually with a QR code that leads the buyer to assembly instructions. Assembly instructions can also be found here: <https://toolgauge.net/wp-content/uploads/2020/04/DtM-Face-Shield-Instructions-Best-Practices-Blog.pdf>

**WHAT CLEANING MATERIALS CAN I USE WITHOUT DAMAGING THE FACE SHIELD?** The DtM Face Shield can be disinfected or cleaned with the following products listed in preferential order: Super Sani-Cloth, 10% chlorine bleach solution (\*may fog transparent shield over time), CaviWipes, soap and water.

**IS THERE MORE THAN ONE SIZE OF THE DtM FACE SHIELD?** The DtM Face Shield comes in a one size fits all, with an adjustable headband strap. Dimensions of the DtM Face Shield are 12" in length and 8" in width.

Submit Orders or Product Questions to:

[SALES@TOOLGAUGE.NET](mailto:SALES@TOOLGAUGE.NET)

This product has not been FDA cleared or approved. This device is being offered for the duration of the FDA Emergency Use Authorization for Face Shields (9 April, 2020). This product is only authorized for the duration of the declaration that circumstances justifying the authorization of emergency use under Section 564(b)(1) of the Act, 21 USC 360bbb-3(b)(1) unless the authorization is terminated or revoked sooner. The use of this DtM face shield should always come secondary to existing PPE equipment, standards, and protocol options if available. The decision to implement this device should be made with consultation of the corresponding institution's occupational health and infection control departments and in line with CDC Strategies for Optimizing the Supply of Eye Protection.